

Blenderized Diet Made Simple



+



Real Food Blends®

The only 100% real food meals that are ready-to-feed for people with feeding tubes.

U Deliver Medical

Bolee® Bag & Bolink® Large Cap
Clean, easy, tube feeding nutrition delivery system.



Together

Your patients get easy-to-deliver nutritious meals and a snack.

Making Tube Feeding Easier Together

Real Food Blends

We bring real food and nutritional variety to people with feeding tubes.

Our meals and snack offer a variety of proteins, fruits, vegetables, whole grains and fats from real foods, and can be used as a primary source of nutrition or in conjunction with commercial formula.

- Convenience of a true blended diet
- 6 meals & 1 snack for kids & adults
- 5–8 real food ingredients in each meal or snack
- Covered by many insurance plans as part of manufactured blenderized natural food (B4149)



U Deliver Medical

We believe the benefits of nutrition will not be achieved if it is hard to deliver tube feeding nutrition.

Our products were designed specifically for the home tube feeder to make tube feeding easy.

- **Bolee Bag** | Reusable 375mL nutrition container
One Bolee Bag eliminates filling seven syringes at every meal
- **Bolink Large Cap** | Connects the Bolee Bag filled with Real Food Blends to a feeding tube or extension set
- Covered by insurance as part of the daily enteral feeding supply kit—gravity fed (B4036 or S9341)

How to use Real Food Blends with U Deliver Medical equipment

1. Pour the Real Food Blends pouch into the Bolee Bag
2. Add additional fluid as needed
3. Bring with you to school, work, appointments or any daily activities
4. Attach the Bolink Large Cap to connect to the feeding tube or extension set at mealtime

Scan the QR code below to watch our easy-to-follow video on how to administer Real Food Blends using a Bolee Bag.



U Deliver Medical
Tube Feeding Made Easy™

REALFOOD
BLENDS®

Bolee Bag is a trademark of U Deliver Medical. For any questions about the Bolee Bag, including its use and cleaning, refer to your product packaging or www.udelivermedical.com.

Real Food Blends is part of Nutricia North America
©2023 Nutricia North America
©udelivermedical 2023

For bolus delivery, keep the syringe tip clean by using the Bolee Bag to fill it.

- Pour the Real Food Blends pouch into the Bolee Bag
- Add additional fluid as needed
- Attach the NeoMed ENFit Transfer lid to the Bolee Bag
- Attach the ENFit bolus syringe to the transfer lid and pull back the plunger to fill

This keeps the syringe tip clean by eliminating the need to dip the syringe into the pouch